


QUICKSTART GUIDE



SETUP

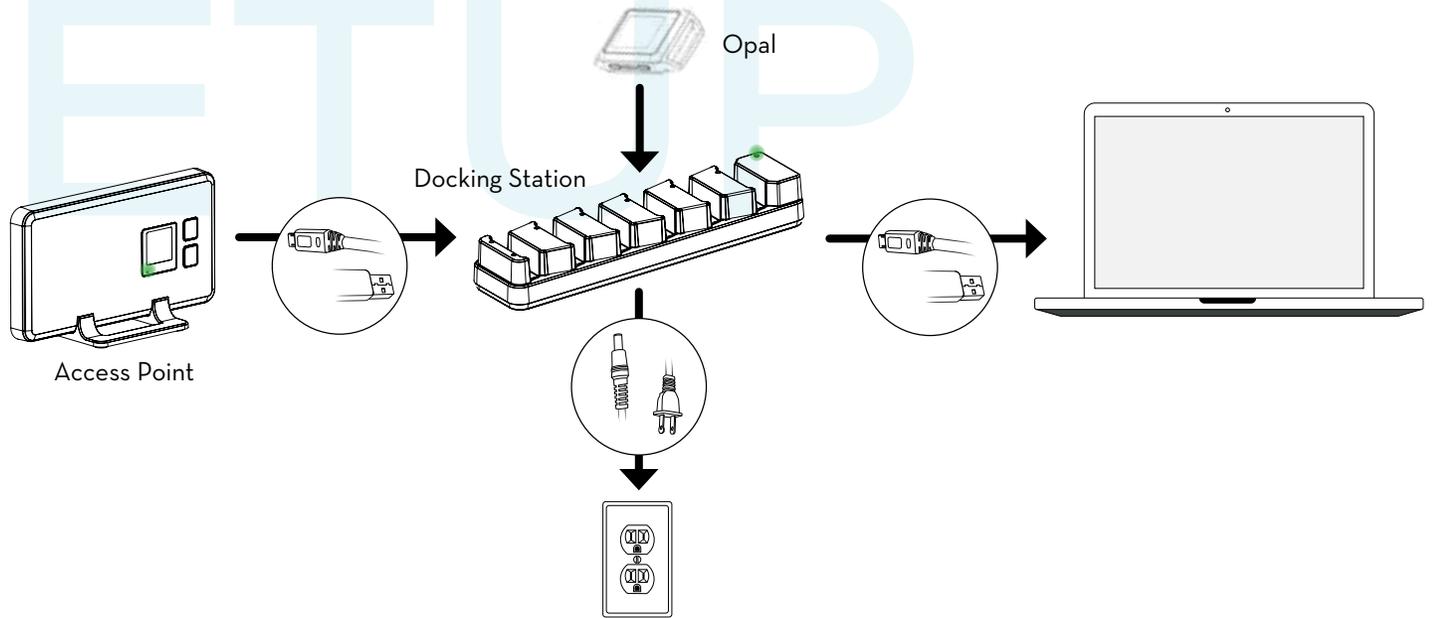


- 1 Connect the Access Point to the Docking Station using a Micro USB cable
- 2 Connect the Docking Station power supply to the Docking Station, and plug it in to a power outlet.
- 3 Connect the Docking Station to your computer using a Micro USB cable.
- 4 Insert the Opals into the Docking Station, in any order, with the port facing down.

Tip: The Docking Station and Access Point should have a green light when powered on.

CONFIGURE

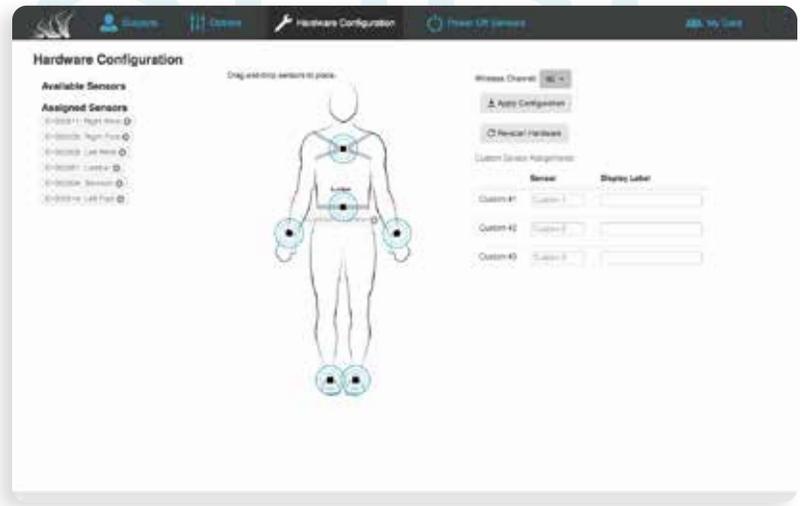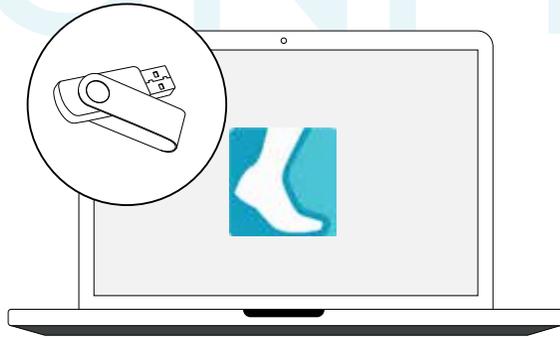

- 5 Install the Mobility Lab software provided on the USB flash drive.
- 6 Open Mobility Lab, name your Subject Group, and click the “Hardware Configuration” tab.
- 7 Assign Opals to a body site location by dragging and dropping from the Available Sensors.
- 8 Click “Apply Configuration.”
- 9 Undock the Opals and confirm that they are blinking green.

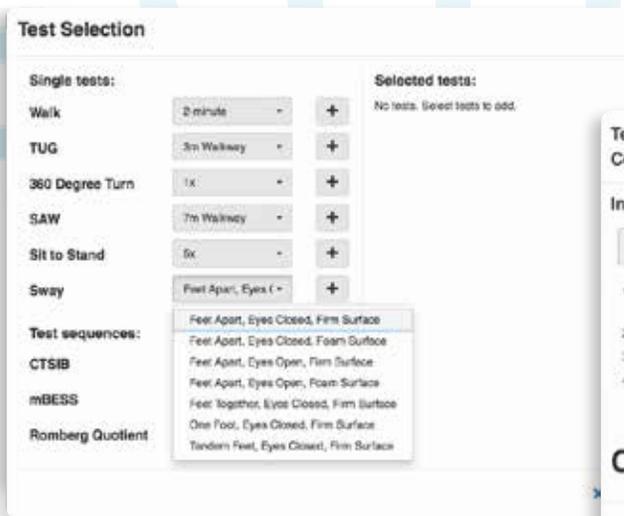

TESTING

- 10 Click the “Subjects” tab and click “+ New Subject” to create a new subject.
- 11 Select the Subject, click “New Test,” and click **+** to add the test(s) you wish to run.
- 12 Select “Next” to continue. Strap the Opals to the appropriate body sites, and then select “Next” to continue.
- 13 Read the Subject and Administrator instructions, and click “Record” to begin the test.

Tip: Your Subject should remain still during the 3 second countdown at the beginning of the tests.