




Moveo Explorer

QUICKSTART GUIDE

Version 2
November 2020



SETUP

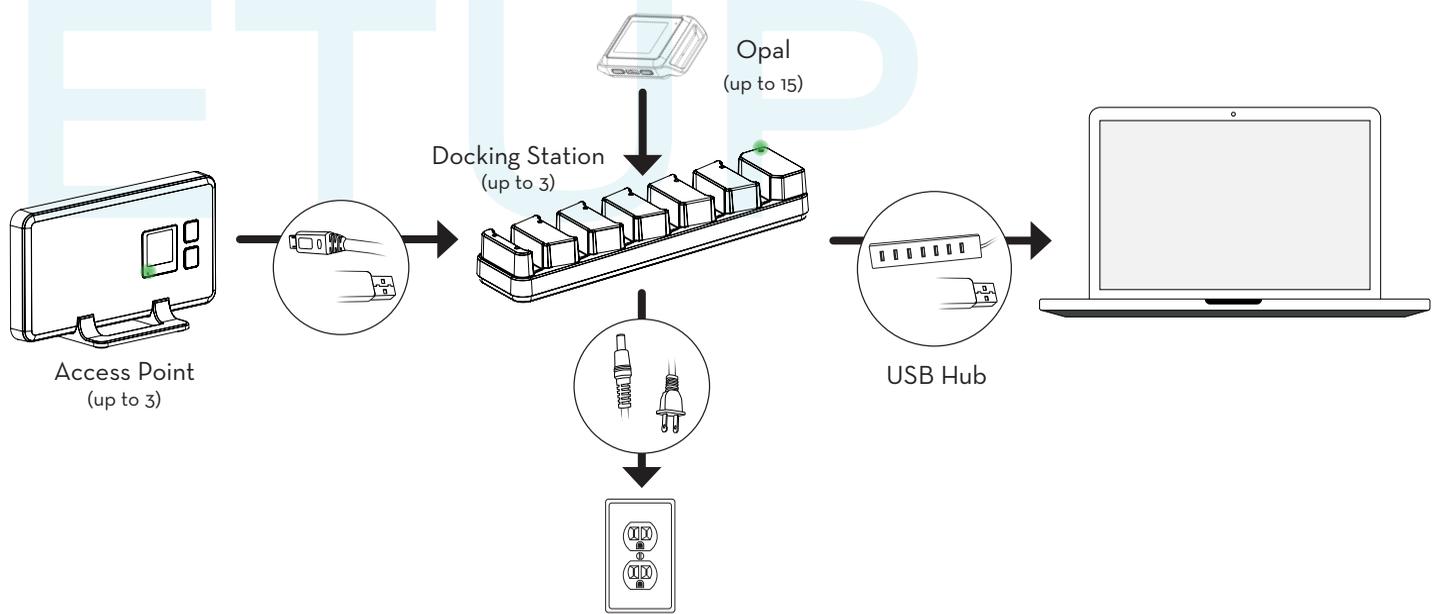

- 1 Connect the Access Point(s) to the Docking Station(s) using a Micro USB cable.
- 2 Connect the Docking Station power supply to the Docking Station(s), and plug it in to a power outlet.
- 3 Using a Micro USB cable, connect the Docking Station(s) to the USB Hub or to the computer directly.
- 4 Insert the Opals into the Docking Station, in any order, with the port facing down.

Tip: The Docking Station and Access Point should have a green light when powered on.

CONFIGURE

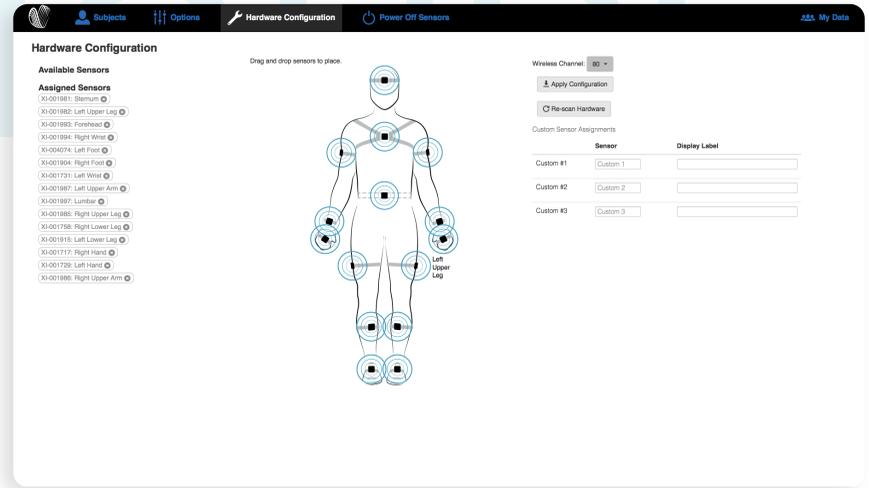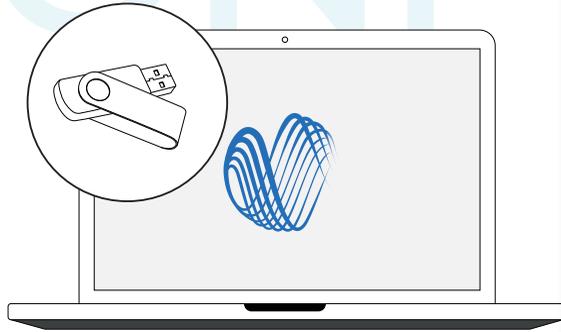

- 5 Install the Moveo Explorer software provided on the USB flash drive.
- 6 Open Moveo Explorer, name your Subject Group, and click the “Hardware Configuration” tab.
- 7 Assign Opals to a body site location by dragging and dropping from the Available Sensors.
- 8 Click “Apply Configuration.”
- 9 Undock the Opals and confirm that they are blinking green in unison.

TESTING

TESTING

TESTING

Test Selection

Single tests:

Walk	2-minute	+
Free Form	Free Form	+
TUG	3m Walkway	+
360 Degree Turn	1x	+
SAW	7m Walkway	+
Sit to Stand	5x	+
Sway	Feet Apart, Eyes	+

Test sequences:

- Feet Apart, Eyes Closed, Firm Surface
- Feet Apart, Eyes Closed, Foam Surface
- Feet Apart, Eyes Open, Firm Surface
- Feet Apart, Eyes Open, Foam Surface
- Feet Together, Eyes Closed, Firm Surface
- One Foot, Eyes Closed, Firm Surface
- Tandem Feet, Eyes Closed, Firm Surface

CTSIB

mBESS

Romberg Quotient

Selected tests:
No tests. Select tests to add.

[Close](#)

Test: Free Form
Condition: Free Form

Instructions:

Subject **Administrator**

- Stand in an upright pose, with your feet directly beneath your hips, and your toes pointed straight forward. Stand as tall as possible, look straight ahead, and keep your arms straight with your palms facing your sides.
- Remain still until you hear the long tone
- Begin movement as instructed by the test administrator

Countdown: 3 s

[Record](#) [Skip](#)

- 10 Click the “Subjects” tab and click “+ New Subject” to create a new subject.
- 11 Select the Subject, click “New Test,” and click **+** to add the test(s) you wish to run.
- 12 Select “Next” to continue. Strap the Opals to the appropriate body sites, and then select “Next” to continue.
- 13 Read the Subject and Administrator instructions, and click “Record” to begin the test.

Tip: Your Subject should hold the calibration pose during the 3 second countdown for proper calibration at the beginning of the tests.